

# How to wear the Comfort-Air NX Series Respirator

## Donning



Hold the mask with your non-dominant hand and the straps with the dominant hand.



Bring the mask to your face and put the straps over your head. Adjust the fitting by pulling the straps on your mask.



Buckle the lower strap behind your head.

## Perform a Seal Check.

Positive pressure check



1. With the palm of your hand, cover the front faces of the filter holder - exhale gently.

2. A successful check is when the facepiece is slightly pressurized before pressure causes outward leakage.

3. Adjust the facepiece and straps if immediate outward leakage is detected.

## Doffing



Unbuckle the lower strap.



With your non-dominant hand support the mask.



Then with your dominant hand pull the straps upward and outward from your head.

## Replacing the filter

As per CDC NIOSH guidelines the filters should be replaced when:

1. The filter is visibly soiled.
2. The cartridge is broken.
3. Or when it is notable harder to breathe through.

## Maintenance



Disassemble the respirator and remove the cartridges.



Clean the respirator with soap and water and air dry.



Wipe the filter cartridge with an alcohol wipe or proper disinfectant.



Place in paper bag and allow the virus to dissipate overnight and reuse the next day.